OM-SH0101E 001

External Coolant Straight Handpiece  
for foot surgery and oral surgery

# Ti-Max X

X-SG65L / X-SG65

OPERATION MANUAL

MADE IN JAPAN

## 1. User and Indications for Use

User: Dentists and surgeon for the intended use

Indications for Use: This device is intended for use by qualified personnel in Dental surgery and other bone surgery (cutting, drilling, reaming, decorticating and smoothing) in a variety of orthopedic surgical procedures performing for foot and ankle surgeries to reduce foot deformities such as Metatarsal resection, Arthroplasty, Arthrodesis, Removal of bony bump, Calcaneal osteotomies.

## 2. Precautions for Handling and Operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 <b>WARNING</b>	<b>Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.</b>
 <b>CAUTION</b>	<b>Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.</b>
<b>NOTICE</b>	<b>General product specification information highlighted to avoid product malfunction and performance reduction.</b>

## **WARNING**

- Immediately after a treatment (within 1 hour), perform maintenance and then store the handpiece. Failure to properly maintain the handpiece may lead to overheating, causing infection, burn injuries or product failure. (Refer to "7. Post-use Maintenance")
- Do not rotate the motor when the Bur Lock Ring is in the OPEN position, or when a bur is not mounted in the chuck. Either situation may cause burn injuries due to sudden heat generation. In addition, the motor insert could be damaged which may lead to difficulty in connecting and disconnecting the handpiece.
- Ball bearings, in time, will become worn out and will require replacement. Be sure to inspect the handpiece before use (Refer to "5. Checking the Handpiece Before Each Use"). If abnormal vibration or noise are found, the bearings or other internal parts may require replacement. Damaged parts may lead to the handpiece to overheat, causing burn injuries. If overheating is suspected, stop using the handpiece immediately and contact your Authorized NSK Dealer.

## CAUTION

- Read this Operation Manual before use to fully understand the product functions and file for future reference.
- When operating the handpiece, always consider the safety of the patient.
- Do not attempt to disassemble the handpiece nor tamper with the mechanism.
- Check the handpiece for vibration, noise and overheating before use. If any abnormalities are found, stop using the handpiece immediately and contact your dealer.
- Should the handpiece function abnormally, cease operation immediately and return the handpiece to the dealer for repair.
- Do not allow any impact on the sheath. Do not drop the handpiece.
- Always keep bur shanks clean. Dirt or debris in a chuck could cause poor bur concentricity or low chuck retention force.

### 3. Connection / Disconnecting the Handpiece and Motor

(1) Connecting

Push the handpiece on to the motor taking care to keep it in alignment.

< X-SG65L only >

Twist the handpiece until it locks into position with a clicking sound.

(2) Disconnecting

Hold the motor and the handpiece, and pull them apart.

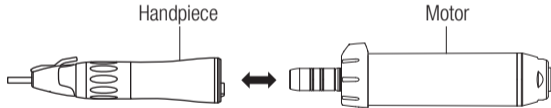
Fig. 1

## 4. Mounting and Removing Bur

### (1) Handpiece bur

#### Inserting the bur

- 1) Turn the Bur Lock Ring in the direction of mark [O] until it clicks.
- 2) Insert the bur.
- 3) Turn the Bur Lock Ring in the direction of the mark [C] until it clicks.  
Make sure the Black Dots align (Fig. 2).

#### Removing the Bur

Turn the Bur Lock Ring in the direction of mark [O] until it clicks and remove the bur.

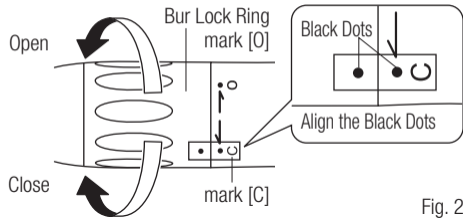
Fig. 2

## (2) Contra-angle bur

### Inserting the bur

- 1) Turn the Bur Lock Ring in the direction of mark [O] until it clicks.
- 2) Insert the supplied Bur Stopper into the chuck. (Fig. 3)
- 3) Insert the contra-angle bur until it stops.
- 4) The bur will be connected by turning the Bur Lock Ring in the direction of mark [C] until it clicks and the Black Dots line up.

### Removing the bur

Turn the Bur Lock Ring in the direction of mark [O] until it clicks and remove the bur.

## ■ Remove the Bur Stopper

- 1) Turn the Bur Lock Ring in the direction of mark [O] until it clicks.
- 2) Insert the supplied Bur Stopper Puller into the chuck. Pull out the Bur Stopper by hooking at the slit of Bur Stopper. (Fig. 4)

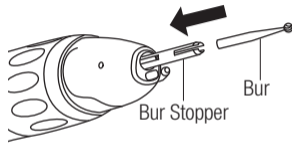
Fig. 3

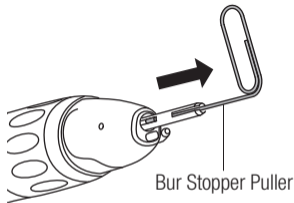
Fig. 4

## CAUTION for Bur Usage

### Heat Generation Problem

- Bur position is improper or it is not mounted.  
--> Mount the bur correctly.
- Bur Lock Ring turned during operation  
--> Do not turn the Bur Lock Ring while the handpiece is rotating.

### Bur Jump Out Problem

- Incorrect mount of the bur/Bur Stopper.  
--> Mount the bur/Bur Stopper firmly.
- Bur Lock Ring turned during operation  
--> Do not turn the Bur Lock Ring while the handpiece is rotating.

- Short shank contra-angle bur mounted.  
--> Do not mount this bur.

### Bur Breakage Problem

- Mount damage or sub-standard burs.  
--> Do not mount these burs.
- Drive motor speed or the manufacturer's recommended bur speed is exceeded.  
--> Do not exceed the drive motor speed or the manufacturer's recommended bur speed.

## 5. Checking the Handpiece Before Each Use

Follow the checklist below before use. If any abnormalities are found, stop using the handpiece immediately and contact your Authorized NSK Dealer.

- 1) Mount the bur (refer to 4. Mounting and Removing the Bur).
- 2) Adjust the handpiece rotation speed in accordance with the allowable maximum speed for the attached bur.
- 3) Rotate the handpiece and check for abnormal rotation, vibration, noise and overheating.

If any of these conditions are found, stop using the handpiece and contact your Authorized NSK Dealer.

- 4) If there are no immediate apparent abnormalities, continue to rotate the handpiece for about 1 minute. Then stop rotation and check for abnormal heating on the outer surface. If abnormal heating is detected, stop using the handpiece and contact your Authorized NSK Dealer.
- 5) Operate the handpiece if no abnormalities are observed.

## 6. External Irrigation Nozzle

Connect the irrigation tube to the External Irrigation Nozzle firmly.

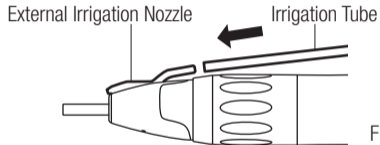
Fig. 5

## 7. Post-use Maintenance for Handpiece

After each patient use, maintain the product as follows.

### **WARNING**

- Immediately after a treatment (within 1 hour), perform maintenance and then store the handpiece. Failure to properly maintain the handpiece may lead to overheating, causing infection, burn injuries or product failure.

### **CAUTION**

- Follow local rules, regulations, and guidelines regarding the reprocessing of devices.
- Perform the following cleaning and disinfection quickly after treatment (within 1 hour) to remove any residue.
- The only sterilization method valid for this product is autoclave sterilization. Other methods such as plasma sterilization or EOG sterilization are not suitable.

## 7-1 Preparations and Pre-use Disinfecting

### CAUTION

- Do not use the following fluids to wipe, immerse or clean the product: strong/super acid water, strong acid/alkaline chemicals, chlorine containing solutions and solvents such as benzine or thinner.
- Do not immerse the product in disinfectant or perform cleaning with an ultrasonic washer. Doing so may cause product failure.

1) Always wear protective gloves, a mask, and protective goggles for safety purposes and to minimize the risk of infection (Fig. 6).

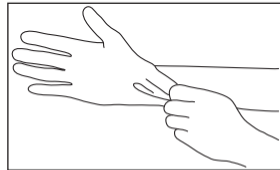
Fig. 6

2) Remove the Bur (Fig. 7).

3) Wipe the exterior of the handpiece clean using a cloth moistened with disinfectant or ethanol with a concentration of 60-90% (Fig. 8).

When using disinfectant, follow the instructions given by the manufacturer of the disinfectant. Use a state-sanctioned disinfectant with proven bactericidal, fungicidal, and virucidal properties. The following disinfectants can be used in the European market.  
Example) MinutenWipes (manufactured by ALPRO MEDICAL GMBH)

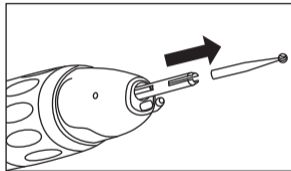
Fig. 7

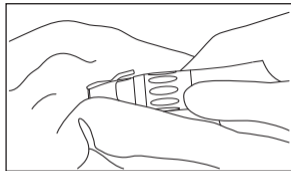
Fig. 8

4) Operate the handpiece while running clean water into the irrigation nozzle via the irrigation tube, for at least 20 seconds to remove fluids in the handpiece (Fig. 9). When operating the handpiece, turn the Bur Lock Ring in the CLOSE position.

After the handpiece rotation has completely stopped, touch the tip of the handpiece to ensure there is NO abnormal heating.

(Water temperature 38°C or less, drinking water quality recommended.)

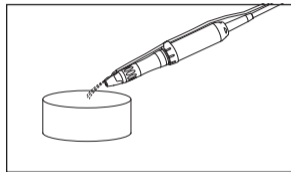
Fig. 9

**⚠ CAUTION**

- If the tip is heating abnormally, stop using the handpiece immediately and contact your Authorized NSK Distributor.

5) Remove the handpiece from the motor (Fig. 10).

6) Bring the handpiece to the decontamination area.

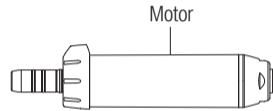
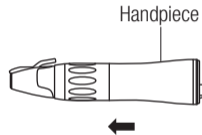
Fig. 10

## 7-2 Cleaning, Disinfecting and Drying

Perform cleaning, disinfecting and drying using either the manual or automatic method.

### Manual Method (Cleaning, Disinfecting and Drying the Exterior)

1) Clean the exterior of the handpiece under running water using a soft brush (non-metallic) (Fig. 11).  
(Water temperature 38°C or less, drinking water quality recommended)

After cleaning the handpiece, confirm that it is clean under appropriate lighting (500 Lux or higher).  
If the handpiece is still dirty, repeat the procedure.

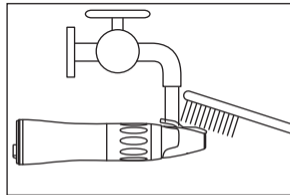
Fig. 11

2) Wipe the exterior of the handpiece clean using a cloth moistened with disinfectant or ethanol with a concentration of 60-90% (Fig. 12).

When using disinfectant, follow the instructions given by the manufacturer of the disinfectant. Use a state-sanctioned disinfectant with proven bactericidal, fungicidal, and virucidal properties.

The following disinfectants can be used in the European market.  
Example) MinutenWipes (manufactured by ALPRO MEDICAL GMBH)

3) If fluid remains on the exterior of the handpiece or fluid has entered the interior of the handpiece, remove it with the help of compressed air from an air gun or similar.

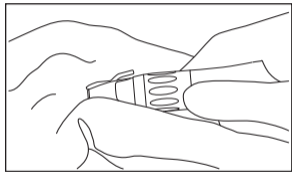
Fig. 12

## Automatic Method (Cleaning, Disinfecting and Drying the Exterior and Interior)

Use a thermo-disinfector that conforms to ISO 15883-1 to perform cleaning, disinfecting and drying.

When using a thermo-disinfector, follow the instructions given by the manufacturer of the thermo-disinfector.

Example) Miele G7891, temperature: 90°C (194°F), duration: 10 min. or vario TD mode

Miele® is a registered trademark of Miele & Cie. GmbH & Co..

After cleaning, disinfecting and drying the handpiece, confirm that it is clean under appropriate lighting (500 lux or higher).

If the handpiece is still dirty, repeat the procedure.

### CAUTION

- Quickly remove the handpiece from the thermo-disinfector after the cleaning, disinfecting and drying cycle is complete (within one hour). Failing to do so may cause corrosion.
- After washing with a thermo-disinfector and prior to lubrication, dry the product until all internal moisture is completely removed. Thermo-disinfector moisture remaining inside the product could reduce the effect of lubrication and could cause corrosion inside the product.

### 7-3 Cleaning and Lubrication

Perform cleaning and lubrication using either the manual or automatic method.

#### Manual Method (Cleaning and Lubricating the Interior)

- 1) Attach an E-Type Spray Nozzle to the nozzle of the PANA SPRAY Plus.
- 2) Shake the can 3 or 4 times, and insert the E-Type Spray Nozzle into the rear of the handpiece while holding the head of the handpiece with a cloth or similar.
- 3) Hold the handpiece and spray for 2 or 3 seconds until PANA SPRAY Plus comes out of the handpiece tip (Fig. 13).

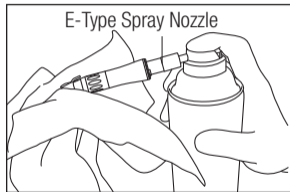
Fig. 13

Repeat lubrication until all foreign material is removed from the tip.

#### **⚠ CAUTION**

- Do not use sprays other than NSK PANA SPRAY Plus. Using sprays other than PANA SPRAY Plus may cause the handpiece to overheat.
- Hold the spray can upright.
- Hold the handpiece firmly to prevent it from slipping when spray pressure is applied.
- Spray lubricant until it expels from the handpiece head.

## NOTICE

- NSK recommends the use of "Spray Mist Absorber" (Y900084) to absorb oil mist expelling out of the handpiece head.

### Automatic Method (Cleaning and Lubricating the Interior)

Use an NSK automatic lubrication system to perform lubrication.

For information on using an automatic lubrication system, see the instruction manual for the automatic lubrication system.

## WARNING

- If blood infiltrates inside a handpiece, an automatic handpiece cleaning and lubrication system may not entirely clean the internal handpiece components and may lead to blood coagulation inside the handpiece. Coagulated blood may cause handpiece failure and overheating, causing burn injuries. To avoid the risk, NSK recommends to immediately clean those handpieces with a non-protein fixing cleaning solution\* before lubricating.

\*Example) WL-clean (manufactured by ALPRO MEDICAL GMBH)

## 7-4 Packaging, Sterilizing and Drying

- 1) Insert the handpiece into the sterilization pouch that conforms to ISO 11607-1 and seal the pouch.
- 2) Perform autoclave sterilization with the following conditions.

Type	Gravity Displacement		Pre-Vacuum (Dynamic Air Removal)
Temperature	132°C(-0/+4°C)	121°C(-0/+4°C)	134°C (-0/+4°C)
Full Cycle Time	15 min. or longer	20 min. or longer	3 min. or longer
Drying Time	30 min. or longer	30 min. or longer	30 min. or longer

### CAUTION

- Follow local rules, regulations and guidelines regarding the reprocessing of devices.
- Immediately after sterilization is complete (within 1 hour), remove the handpiece from the sterilizer. Failure to do so may cause corrosion.
- Do not autoclave the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.

 **CAUTION**

- Clean and lubricate the handpiece prior to autoclaving. If blood remains on the internal surface it can become clotted and cause product failure.
- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- To avoid product failure, do not use a sterilizer that exceeds a cycle temperature of 138°C, including the dry cycle. In some sterilizers, the chamber temperature may exceed 138°C. Contact the sterilizer manufacturer for detailed information about cycle temperatures.
- Do not touch the product immediately after autoclaving as it will be very hot and must remain in a sterile condition.
- Autoclave sterilization is recommended for the product. The validity of other sterilization methods (such as plasma sterilization or EOG sterilization) is not confirmed.

**NOTICE**

- NSK recommends Class B sterilizers as stated in EN 13060.

## 7-5 Storage

Store the product in a clean location without humidity.

### CAUTION

- Store the products in a place avoiding adverse effects from atmospheric pressure, temperature, humidity, direct sun, dust and salinity/sulfur content atmosphere.
- Sterilization is not guaranteed after the sterilization retention period specified by the manufacturer and seller of the sterilization pack has elapsed.
- If the sterilization retention period has elapsed, perform sterilization again with a new sterilization pouch.

## 8. Post-use Maintenance for Reuseable Bur

- 1) Remove the bur from the handpiece.
- 2) Clean the bur under running water (water temperature 38°C or less, drinking water quality recommended).
- 3) Insert the bur into the sterilization pouch and seal the pouch. Perform autoclave sterilization.
- 4) Store the bur after adequately drying the bur.

\*Recommended conditions of the bur, refer to "7-4 Packaging, Sterilizing, and Drying".

### CAUTION

- Conduct maintenance (cleaning, washing and sterilizing) of the bur in accordance with the operation manual for the bur.
- Store the products in a place avoiding adverse effects from atmospheric pressure, temperature, humidity, direct sun, dust and salinity/sulfur content atmosphere.

## 9. Cleaning of Optic Illumination Points

Wipe clean all Optic Illumination Points (Glass Rod) using an alcohol-immersed cotton swab. Remove all debris and oil.

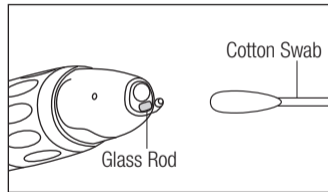
Fig. 14

### CAUTION

- Do not use a sharp tool to clean the Glass Rod. It could damage the glass and reduce the light transmission.

## 10. Periodical Maintenance Checks

Every 3 months perform periodical maintenance checks, referring to the check sheet below. If any abnormalities are found, contact your Authorized NSK Dealer.

Points to check	Details
Rotation	Rotate the handpiece and check for abnormalities such as abnormal rotation, vibration, noise, and overheating.
Bur mounting/removing	Make sure that the bur can be properly mounted/removed. After mounting, make sure that the bur is firmly fixed by pulling and pushing it.

## 11. Specifications

Model	X-SG65L	X-SG65
Gear Ratio	1:1	
Allowable Max Speed	40,000min <sup>-1</sup> (rpm)	
Bur Type	ISO 1797-1 (EN ISO 1797-1) Type 2 Ø2.35mm HP bur and ISO 1797-1 (EN ISO 1797-1) Type 1 Ø2.35mm CA bur	
Bur Applied Length	29.4mm (HP Bur)	
Max. Bur Length	44.5mm (HP Bur)	
Max. Bur Working Diameter	Ø4.0mm	
Irrigation	External Irrigation	
Optic	Glass Rod	-
Use Environment	Temperature : 0 - 40°C, Humidity : 30 - 75%, Pressure : 700 - 1,060hPa	
Transportation and Storage Environment	Temperature : -10 - 50°C, Humidity : 10 - 85%, Pressure : 500 - 1,060hPa	

Use the following bur.

- ISO 1797-1 (EN ISO 1797-1) compliant bur
- 93/42/EEC compliant bur
- Oral Surgery·····Disposable or re-useble bur
- Foot and Ankle Surgery·····Disposable bur

 **CAUTION**

- Be sure to confirm the bur handling and usage in the operation manual.

## 12. Symbol

This product is Autoclavable up to Max.135°C

This product can be washed via thermo disinfectant.

Manufacturer.

0197

Conforms to CE European Directive of "Medical device directive 93/42/EEC."

Authorized representative in the European community.

## 13. Warranty

NSK products are warranted against manufacturing errors and defects in materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose, has been tampered with by unqualified personnel, or has had non NSK parts installed. Replacement parts are available for seven years beyond discontinuation of the model.

## 14. Spare Parts List

Model	Order Code
E-Type Spray Nozzle	Z019090

## 15. Disposing product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

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**NSK Europe GmbH**